

Case Study

Use of the ArcPhix™ Angled Compression Screw for Functional Fusion of the Third Distal Interphalangeal (DIP) Joint



Lloyd Champagne, MD

Arizona Center for Hand to Shoulder Surgery
Phoenix, AZ

Dr. Lloyd Champagne, a graduate of Louisiana State University School of Medicine, completed a plastic surgery along with a hand and microsurgery fellowship while at the University of Pittsburgh. He specializes in plastic and hand reconstructive surgery.

Case Presentation

Patient was a 55-year-old male who was scheduled for DIP joint fusion to his middle finger due to osteoarthritis. A percutaneous approach providing compression and stability was desired to achieve early mobilization and return to daily activities.

Pre-op Plan

Dr. Champagne normally considers fixation with multiple K-wires but wanted to avoid extramedullary hardware due to the possibility of infections. K-wire fixation also can result in unsatisfactory fixation and typically requires multiple follow ups. Dr. Champagne decided to proceed with the ArcPhix angled compression screw using a minimally invasive approach to achieve rigid fixation. The angled design allowed for a simple and reproducible implantation technique with the additional benefits of improved finger dexterity and grip strength achieved by functional flexion.

Operative Findings and Approach

Dr. Champagne used the cup and cone technique to prepare the bones of the middle finger in a manner that allows for good apposition at the desired angle. He aligned the distal and middle phalanges, in a manner typical for standard IM K-wire placement, to insert the guide wire percutaneously into the distal phalanx. Dr. Champagne drilled by passing the cannulated drill over the guide wire to the desired depth of the implant, which was approximately 16 mm into the middle phalanx. He removed the drill and guide wire to insert the ArcPhix angled compression screw into the drill hole at the tip of the distal phalanx.

Dr. Champagne advanced the screw until the apex of the bend was across the DIP joint with the convex side of the screw facing dorsally. With ArcPhix, Dr. Champagne achieved excellent compression across the DIP joint along with good rotational stability. Total surgery time was approximately 25 minutes.

Preoperative



Postoperative



Follow-up

At one week post-op, the patient was recovering well and had full range of motion (ROM) at both the metacarpophalangeal (MCP) and proximal interphalangeal (PIP) joints. At three weeks post-op, radiographs were satisfactory and the patient was allowed to return to his daily activities without any complications or restrictions.

Discussion

Dr. Champagne has been pleased with the ArcPhix angled compression screw because of the minimally invasive approach that allows for immediate mobilization and excellent outcomes achieved by compression and stable fixation. Most importantly, his patients have been pleased with the functional flexion achieved with the bent design of ArcPhix.



SPF70-44-A

Effective: 2024/11

© 2024 Acumed® LLC

www.acumed.net

Acumed Oregon Campus
5885 NE Cornelius Pass Road
Hillsboro, OR 97124
+1.888.627.9957

Acumed Texas Campus
3885 Arapaho Road
Addison, TX 75001
+1.800.456.7779

Acumed Iberica Campus
C. Proción, 1
Edificio Oficor
28023 Madrid, Spain
+34.913.51.63.57

These materials contain information about products that may or may not be available in any particular country or may be available under different trademarks in different countries. The products may be approved or cleared by governmental regulatory organizations for sale or use with different indications or restrictions in different countries. Products may not be approved for use in all countries. Nothing contained on these materials should be construed as a promotion or solicitation for any product or for the use of any product in a particular way which is not authorized under the laws and regulations of the country where the reader is located. Specific questions physicians may have about the availability and use of the products described on these materials should be directed to their particular authorized Acumed distributor. Specific questions patients may have about the use of the products described in these materials or the appropriateness for their own conditions should be directed to their own physician.

ExsoMed™ is a wholly owned subsidiary of Acumed LLC.

Acumed®, ExsoMed™, and ArcPhix™ are registered trademarks of Acumed LLC.